



1. Align the dots if available. Gently insert all of the ampoule lid into the Snapit. The ampoule lid becomes locked in at the neck.



2. With little effort, lever away from the dot to snap off the lid. The lid should remain held in the Snapit.



3. Carefully aim over a sharps bin and eject the lid by pushing down the metal rod with your thumb.

NOTE: If your standard practice requires you to turn the dots away from your body, make sure that both dots are still aligned after turning and before snapping.

IMPORTANT NOTE:

If you do not break away from the dot on the glass ampoule, the glass lid may shatter inside Snapit.

NOTE:

- The lid is propelled out. Aim closely over the sharps bin.
- Only eject ampoule lids into an approved sharps bin.



4. **Clean your Snapit regularly.** Remove the O-ring as shown to disassemble the parts and clean with water or a hospital grade cleaner. Do this in a clean dish to avoid loss of parts. Dry completely then re-assemble.

NOTE:

If the ampoule lid breaks, tap the Snapit over a sharps bin to remove excess glass pieces then rinse under running water before disassembling

Sizes

- If the ampoule lid is too big, it will not fit through the entrance. Use a larger Snapit
- If the ampoule is too small, the Snapit will not hold the top securely. Use a smaller Snapit

Ampoule Size	Snapit Size
1 – 2 ml	REGULAR Most Common Sizes
5—10 ml	
10—15 ml	

Ampoule Size	Snapit Size
5 – 10 ml	LARGE
10—15 ml	
20—25 ml	

Due to the inconsistent variation of tops, this is only a guideline

Disclaimer

It is important to become familiar with these instructions before using your Snapit. Incorrect use may create safety hazards. It may be necessary to clean your Snapit after opening some ampoules. Goods are warranted for 1 year faulty workmanship. Physical damage or loss of parts will void all warranties. Receipt must be presented for all warranty claims.